

Reusable face mask

Washable and reusable mouth-nose protection masks with membrane system

The washable mouth mask is made to protect yourself against harmful particles that can be transported by air. The reusable mouth-nose protection mask is made of medicafabric with a barrier membrane. The used materials comply with the requirements of EN 13795: 2019 "surgical drapes and gowns - requirements and test methods"

The materials of these masks have been used for many years in reusable surgical gowns and drapes. They also have been tested for the resistance to microbial penetration. Corresponding test reports are available.

Advantages of the reusable mouth-nose mask:

- Barrier membrane regains its protective properties after washing at 60°
- reduction of (medical) waste due to the reusable nature of the fabric

Order codes	Materials
TX240177B7NB50 – Light Blue	100% polyester laminate with PolyTetraFluorEthylene (PTFE)
TX240177B6LG50 – Green	100% polyester laminate with polyurethane membrane
TX240177B6LY50 – Grey	100% polyester laminate with polyurethane membrane
TX240177B8LW50 – White	100% polyester laminate with PolyTetraFluorEthylene (PTFE) / Polyurethane (PU) membrane



Available in the colors light blue, green, grey and white.



Hold the mask by the ear loops and place a loop around each ear



Pull the bottom of the mask over your mouth and chin




Avoid touching the front of the mask when wearing

Elastics may contain latex
Sewing thread in 100% polyester

Order quantities: packages of 50, 100, 500, 750, 1000 and 2000 pieces

In case of questions please contact the Customer Service at sales@eurotape.com

- Minimum Order Quantity 50 pieces
- Wash before each use
- Maximum number of washing cycles: 45 to a maximum of 70 times
- Recommended wash prescriptions

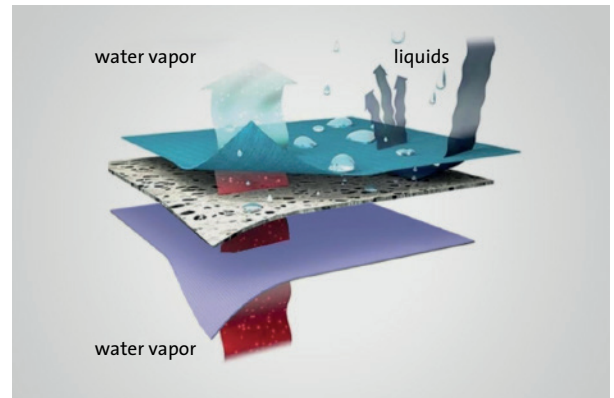

Washing instructions



Water temperature 60°C



Do not bleach



Low heat



Do not iron



Do not tumble dry

Storage instructions

Store at a temperature that does not exceed 25°C with a maximum relative humidity of 80%. Keep away from light sources. Do not store longer than 1 month at average conditions higher than indicated.

Additional information

The materials comply with EN 13795: 2019 technical standards "surgical drapes and gowns - requirements and test methods" for performance requirements and ISO 10993-5: 2009 "Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity" for the harmlessness requirements. The mask can be used for a maximum of 4 hours at a time. The mask is not a medical device. The mask is not a personal protection device.

Manufactured by

Creazoni Futura S.r.l.
www.creazonifutura.com
Italy



Distributed by

Eurotape BV
Oostergracht 2
3763 LZ Soest
The Netherlands

Tel +31 (0)35 609 10 20
sales@eurotape.com
www.eurotape.com

EUROTAPÉ

Connecting with care